



ULTRAEZ™ DESENSITIZING GEL TRAYS INSTRUCTIONS

PRODUCT DESCRIPTION:

UltraEZ™ works instantly to alleviate discomfort and sensitivity from teeth whitening. UltraEZ™ is also ideal for treating patients experiencing sensitivity from toothbrush abrasion, root exposure and deep cleanings (gum treatment). Formulated with 3% potassium nitrate and a 0.11% fluoride ion, UltraEZ™ stays in contact with the teeth for fast relief while reducing sensitivity.

HOW TO USE:

STEP 1: Remove the desensitizing trays from the packaging.

'U' - upper tray

'L' - lower tray

STEP 2: Position tray on teeth.

STEP 3: Bite firmly then gently suck down on the tray for 2 seconds. Do not press against teeth as it pushes the gel against the gums.

STEP 4: Remove outer tray leaving the white inner tray on the teeth.

STEP 5: Repeat with the opposite arch if needed.

STEP 6: Wear for 15 minutes to 1 hour per day or as directed by your dentist.



NOTES:

- Do not eat, drink, or use tobacco products while trays are in place. Do not use tray overnight.
- Can be used before and after teeth whitening.
- Keep out of heat and/or sunlight.
- Brush off and rinse any excess material at insertion and removal. DO NOT SWALLOW.
- Contact us if you need additional information and or help. We are here for YOU!